

SAFETY INFORMATION

SAFETY WARNINGS

- Ensure maintenance and checks below are followed.
- Non self-propelled Chairs should be used when a carer is present.
- Never use your FREEWAY Shower / Toilet Chair without ensuring that the black retaining clips secure the seat in the "locked" position.
- Armrests and backrests are detachable to aid in transfer and storage only; both Armrests and Backrests should be fitted to the chair when in use.
- Always ensure that the hand nuts at the base of the Backrests are tight at all times during the use of the Shower / Toilet Chair.
- On self-propelled models, when engaging the brake on the large wheels, ensure that the lever is fully depressed to its "stop."
- When using the chair over a toilet, always ensure that the toilet lid & seat are up.

STABILITY

- YOUR FREEWAY SHOWER / COMMODE CHAIR is very stable in its design. Abuse however, could cause the safety to be compromised.

Therefore:

- Do not stand on the Footrests, or allow the Footrests to be stood upon.
- During transfer, engage all brakes.
- Do not sit upon, or allow to be sat upon, the Armrests of the Chair.

CLEANING

- Wipe clean and dry as much as possible after use.
- Once a week or more often as necessary, clean with an all-purpose cleaning fluid or mild disinfectant, rinse, then dry as much as possible.

MAINTENANCE

BEFORE EACH USE:

- Check backrest nuts and seat securing clips are in position, Footrests are securely on spigots armrest locking nuts are in position and brake levers are secure.

EACH WEEK:

- As daily check
- Check Castors turn without obstruction. Apply brakes and test function.
- Check wheels on self-propelling models are free running but without excessive play, and that the nut on the axle bolt is tight.
- Check seat for damage to vinyl or base that might allow ingress of water.
- Check backrest vinyl is secure and free from damage.

EVERY YEAR:

- The chair should be serviced every twelve months to check fittings for wear, and to make any adjustments necessary.

**DO NOT DROP SEAT
DO NOT JET WASH CASTORS OR SEAT**

**There is a large selection of optional extras that can be fitted to FREEWAY Chairs.
See your agent for details**

FREEWAY