

### General safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

#### 1.0 Intended Use

The Wheeled Overbed/ Divan Overbed Tables are intended for use when sitting in either a chair or bed, to provide an adjustable height/angle table surface for reading, eating etc. The divan version simply slides under a divan bed whilst the wheeled version features four lockable castors. **Please note;** these products are not designed to be weight bearing and must not be used as a walking aid or trolley.

#### 2.0 Safety Precautions

- Inspect regularly for wear and damage.
- Do not overload table top (10Kg Max).
- Do not attempt to use as an aid to walking.
- Check hand wheels and e-clips are secure at regular intervals.

#### 3.0 Unpacking

Remove packaging and check all parts carefully before assembling. If any parts are damaged or missing, see contents list below, do not assemble the product and contact your supplier immediately

| <u>Contents</u>                    | <u>Qty.</u> |
|------------------------------------|-------------|
| Base Frame*                        | 1           |
| Leg with pin e-clip (white finish) | 2           |
| Angled Tube (chrome finish)        | 2           |
| Table top                          | 1           |
| Handwheel                          | 2           |

\*L17516 With castors, Fig.1

\*F19959 Divan, without castors. Fig.2






Figure:1  
L17516



Figure:2  
F19959

#### 4.0 Assembly

- 1 – Table top assembly:** Fit the angle tube into the black plastic saddle clamps on the underside of the table top, by pressing the spring button clip inwards and pushing the tube through both clamps. Repeat for the other tube, see fig.3a & 3b
- 2 – Base assembly:** Fit the legs into the base and secure using the pin e-clips, see fig.5
- 3 – Engage the handwheels into the tubes, do not fully tighten as this will prevent the angled tubes from sliding into the legs**
- 4 - Slide the two angled tubes attached to the table top into the base legs see, fig. 4**
- 5 - Adjust the height and angle of the table top as required and tighten the hand wheels clockwise to secure see, fig. 4**

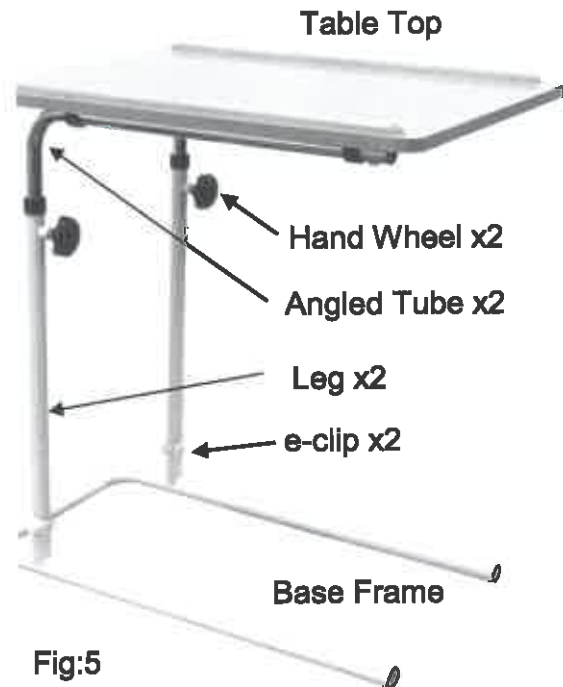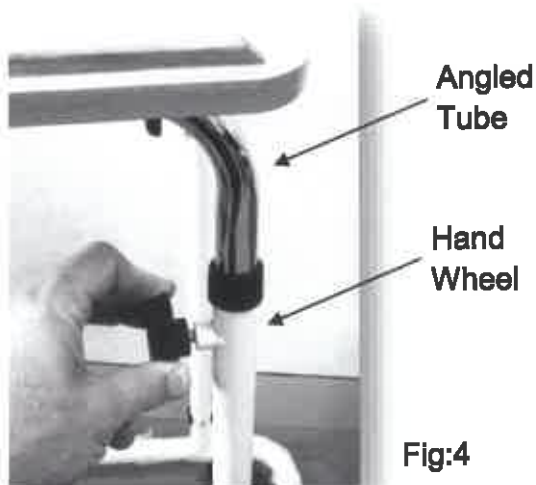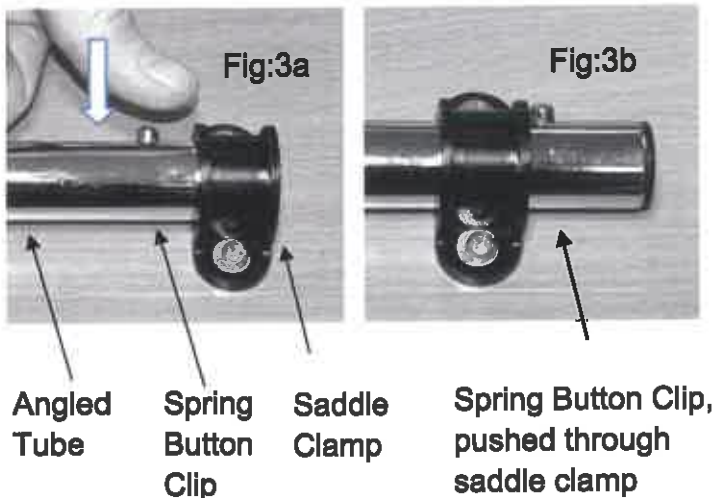

#### 5.0 Cleaning & Maintenance

Clean regularly, wiping down with a mild detergent /disinfectant solution.

Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will damage surfaces and invalidate any warranty. Check all parts regularly for signs of wear or damage, withdraw from use if you see or suspect a fault.

#### 6.0 Product Disposal

Clean product prior to disposal. Product is manufactured from metal, wood composite and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

#### 7.0 Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk)

**PASS  
QC 001**

ULF1995W/LT7516/0178/1a3U