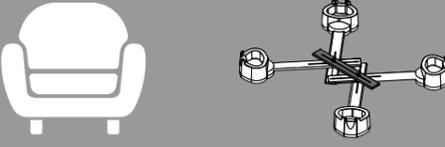
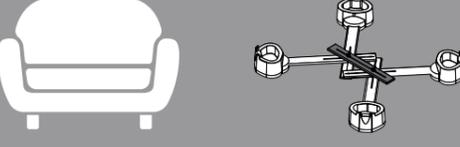
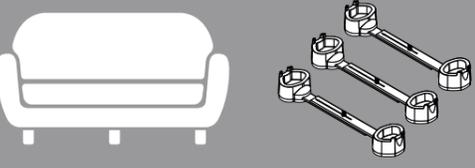
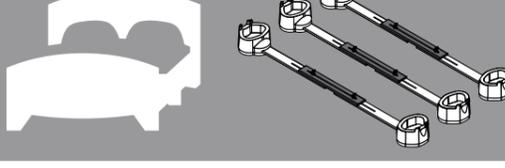


INSTRUCTIONS FOR USE

i TRIED AND TESTED BY OCCUPATIONAL THERAPISTS FOR DECADES, THE LANGHAM MULTI PURPOSE RAISER IS DESIGNED FOR CHAIRS, SETTEES AND BEDS TO GIVE THE HIGHEST LEVEL OF STABILITY.

! DEVICE TO BE USED IN STATIC MODE ONLY. EXTRA CARE SHOULD BE TAKEN TO AVOID SUDDEN OR UNDUE MOVEMENT. IT IS THE RESPONSIBILITY OF THE OWNER/ USER TO CARRY OUT AN INITIAL AND REGULAR LOCAL RISK ASSESSMENTS. DO NOT USE WITH ANY OTHER DEVICE UNLESS STATED. TAKE CARE TO AVOID ENTRAPMENT OF FINGERS IN ADJUSTMENT SLOTS. WARNING: POTENTIAL TRIP HAZARD IF RAILS PROTRUDE THE OUTER FOOTPRINT OF THE CHAIR. CHECK FOR STABILITY BEFORE EACH USE. ALL CASTORS/ LEGS MUST BE SUPPORTED BY A RAISER UNIT. FURNITURE MAY BE HEAVY, TAKE CARE WHEN FITTING THE RAISER.

i REFERENCE INFORMATION

ARMCHAIRS WITH 4 CASTORS/ LEGS	2 MPR MAIN UNITS	RAISE REQUIRED	CLIP-ON HEIGHT SECTIONS
	1 SPREADER BAR	3 inch (7.5cm)	0
		3 ¾ inch (9.5cm)	4
		4 ½ inch (11.5cm)	8
		5 ¼ inch (13.5cm)	12
		6 inch (15.5cm)	16
SMALL SOFAS WITH 4 CASTORS/ LEGS	2 MPR MAIN UNITS	RAISE REQUIRED	CLIP-ON HEIGHT SECTIONS
	1 SPREADER BAR	3 inch (7.5cm)	0
		3 ¾ inch (9.5cm)	4
		4 ½ inch (11.5cm)	8
		5 ¼ inch (13.5cm)	12
		6 inch (15.5cm)	16
LARGE SOFAS WITH 6 CASTORS/ LEGS	3 MPR MAIN UNITS	RAISE REQUIRED	CLIP-ON HEIGHT SECTIONS
	0 SPREADER BARS	3 inch (7.5cm)	0
		3 ¾ inch (9.5cm)	6
		4 ½ inch (11.5cm)	12
		5 ¼ inch (13.5cm)	18
		6 inch (15.5cm)	24
SINGLE BED WITH 4 CASTORS/ LEGS	2 MPR MAIN UNITS	RAISE REQUIRED	CLIP-ON HEIGHT SECTIONS
	0 SPREADER BARS	3 inch (7.5cm)	0
		3 ¾ inch (9.5cm)	4
		4 ½ inch (11.5cm)	8
		5 ¼ inch (13.5cm)	12
		6 inch (15.5cm)	16
DOUBLE BED WITH 6 CASTORS/ LEGS	3 MPR MAIN UNITS	RAISE REQUIRED	CLIP-ON HEIGHT SECTIONS
	3 SPREADER BARS	3 inch (7.5cm)	0
		3 ¾ inch (9.5cm)	6
		4 ½ inch (11.5cm)	12
		5 ¼ inch (13.5cm)	18
		6 inch (15.5cm)	24



PH 5 - 9

CLEANING INSTRUCTIONS

WIPE DOWN WITH A NON-ABRASIVE CLOTH AND SOAPY WATER. DEVICE IS FULLY AUTOCLAVABLE.

VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK

 [gordonellis.healthcare](https://www.facebook.com/gordonellis.healthcare)
 @EllisHealthcare



GORDON
ELLIS
& CO

LANGHAM[®]

MULTI PURPOSE RAISER

Welcome to your new Gordon Ellis product.
We hope that it proves a useful aid in your daily life.



EN ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE EC REP (EU CUSTOMERS) OR THE MANUFACTURER (UK & NON-EU CUSTOMERS) - CONTACT DETAILS BELOW



SEE REVERSE FOR FITTING INSTRUCTIONS
AVAILABLE IN THE FOLLOWING LANGUAGES:
EN




MAXIMUM USER WEIGHT
PER SET OF FOUR

Your **LANGHAM MULTI PURPOSE RAISER** was specifically developed by our UK Design Team who have over 30 years experience in raising furniture, coupled with feedback received from Occupational Therapists. This is the first linked raiser designed to specifically raise chairs, settees, and beds equally, with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.

i MULTI PURPOSE RAISER
LP/MPR-BK

i SPREADER BAR
LP/MPR/SB

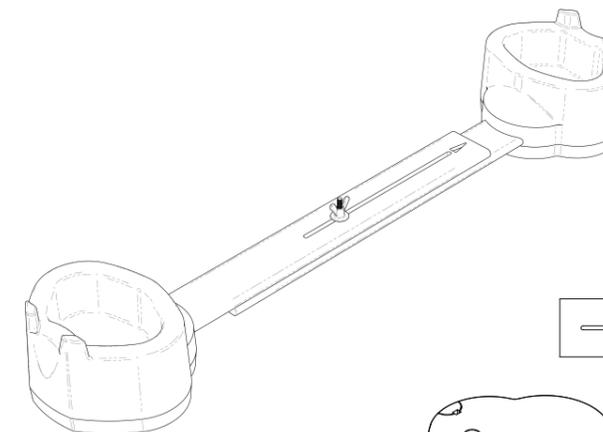

i XL SPREADER BAR
LP/MPR/4

i CLIP-ON BASE
LP/MPR/BASE-BK



IFU-REF LP_MPR-BK/1120-1

PART REF LP/MPR-BK

EC REP

MD



DESIGNED
& MADE
IN THE UK



5
PP



GORDON ELLIS & CO.
TRENT LANE
CASTLE DONINGTON
DERBY. ENGLAND DE74 2AT


AUTHORISED REPRESENTATIVE
ADVENA LTD, TOWER BUSINESS CENTRE,
2ND FLOOR, TOWER STREET, SWATAR, BKR
4013 MALTA

VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK

 [gordonellis.healthcare](https://www.facebook.com/gordonellis.healthcare)
 @EllisHealthcare




FITTING TO ARMCHAIRS - 2 X MPR MAIN UNITS + 1 X SPREADER BAR + CLIP ON HEIGHT SECTIONS AS PER DESIRED RAISE HEIGHT

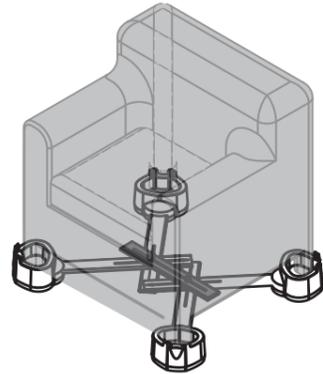


1. Place the MPR Units on the floor behind the chair, attach the Spreader Link Bar running across the chair side to side.
2. If Clip-on Height Sections are to be used they should be clipped on to the Corner Supports; it is important that all corners are of equal height.
3. Loosen the Wing Nuts, and position the front Corner Supports as far out as possible.
4. Adjust the rear Corner Supports so that they are the same width as the furniture.
5. Carefully tilt the furniture forwards and slide each of the rear Corner Supports under the rear corners of the chair.
6. Lower the chair on to the Corner Supports and ensure that the castor is sitting inside the support well.
7. Move to the front of the chair and carefully tilt it backwards. Swing the front Corner Supports round and lower the furniture on to them; again ensure the castor is located in the well.
8. Go to each corner and rotate the Support to make sure it is snug against the furniture.
9. Once each corner is positioned with the minimum intrusion and is tight to the furniture tighten the Wing Nut.
10. Before first use check the furniture is stable on the MPR.

FITTING TO LARGE SOFAS - 1 X CORNER SUPPORT FOR EACH LEG/ CASTOR + CLIP ON HEIGHT SECTIONS AS PER DESIRED RAISE HEIGHT

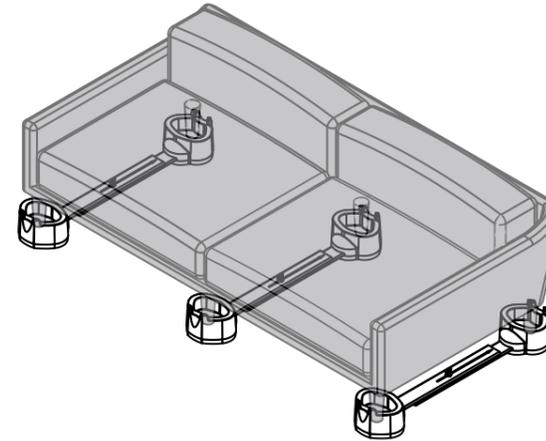


1. For extra large settees it is preferable to use three independent MPR Units in the same way you would raise a single divan bed with a central support.
2. If Clip-on Height Sections are to be used they should be clipped onto the Corner Supports; it is important that all corners are of equal height.
3. Place one Corner Support under each arm, running front to back and one MPR Unit as a central support in the middle.
4. If the furniture has any additional points of contact with the floor they too must be supported.
5. Make sure each Corner Support is tight up to the settee and tighten the Wing Nuts.

FITTING TO SMALL SOFAS - 2 X MPR MAIN UNITS + 1 X SPREADER BAR + CLIP ON HEIGHT SECTIONS AS PER DESIRED RAISE HEIGHT

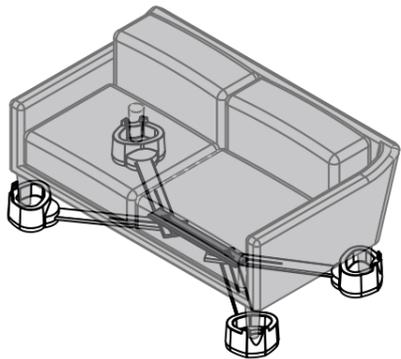

1. Place the MPR Unit on the floor behind the settee.
2. Attach extended Spreader Link Bar to each MPR unit (Part No K203ft6).
3. If Clip-on Height Sections are to be used they should be clipped onto the Corner Supports; it is important that all corners are of equal height.
4. Loosen the Wing Nuts, and position the front Corner Supports as far out as possible.
5. Adjust the rear Corner Supports so that they are the same width as the furniture.
6. Carefully tilt the furniture forwards and slide each of the rear Corner Supports under the rear corners of the furniture.
7. Lower the settee on to the Supports and ensure that the castor is sitting inside the well.
8. Move to the front of the settee and carefully tilt it backwards. Swing the front Corner Supports round and lower the furniture on to them; again ensure the castor is located in the Well.
9. Go to each corner and rotate the Support to make sure that it is snug against the furniture.
10. Once each corner is positioned with the minimum intrusion and is tight to the furniture tighten the Wing Nut.
11. Before first use check the furniture is stable on the MPR.

FITTING TO BEDS - 1 X CORNER SUPPORT FOR EACH LEG/ CASTOR + SPREADER BAR DEPENDING ON BED WIDTH + CLIP ON HEIGHT SECTIONS AS PER DESIRED RAISE HEIGHT

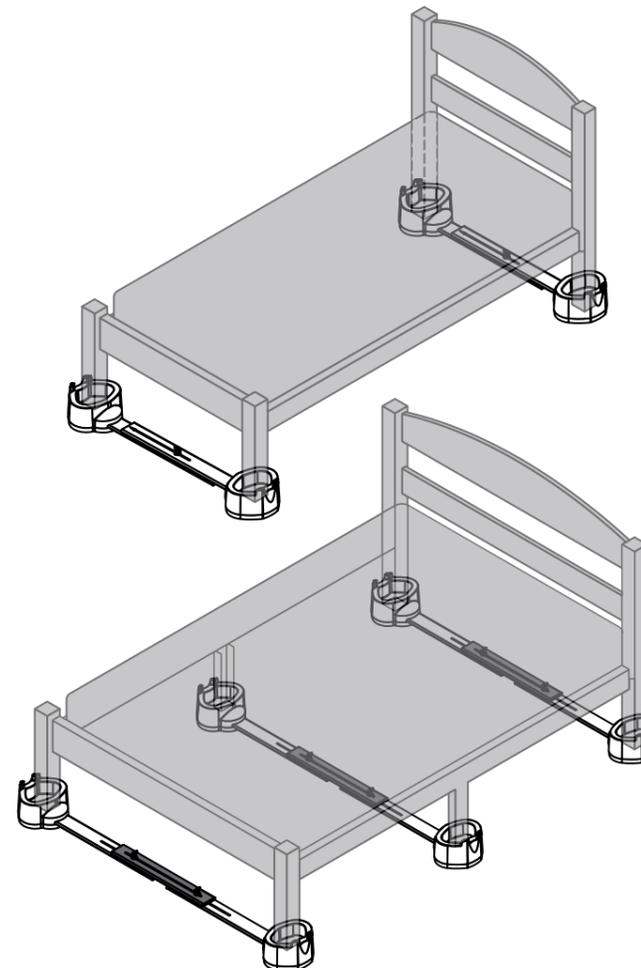

1. If Clip-on Height Sections are to be used they should be clipped on to the Corner Supports; it is important that all corners are of equal height.
2. Place one MPR unit parallel to the foot of the bed.
3. Loosen Wing Nuts and adjust to same width as the bed. For double beds use Spreader Link Bars to extend width.
4. Lift bed and place corners on to the supports, ensure the castor is sitting inside the Well.
5. Push Corner Supports tight up against the bed, rotate each Support to give best possible fit. If using an MPR as a central support ensure it is pushed up tight to sides of bed.
6. Repeat for the opposite end of the bed. You must use a Support wherever you have a castor.
7. Tighten all Wing Nuts and check bed is stable on MPR before use.