Set the height of the Medeci MFR1

The Medeci MFR1 is supplied set to its lowest height (figure 3). To increase the height, rotate the base (a) relative to the body (c). Each complete rotation increases the height by 5mm (0.25 inches). There is a height guide (b) on the side of the body. The numbers on the height guide are in millimetres and give the approximate raising above the lowest setting. **Do not attempt to raise above the MAX mark.** The overall height is adjustable from 100mm (4 inches) to 176mm (7 inches). The effective castor height adjustment is 50mm (2 inches) to 127mm (5 inches). The height of each raiser can be adjusted separately by making a part rotation of the base to allow for uneven floors.

Attaching and removing the Medeci MFR1 furniture raiser.

The Medeci MFR1 is supplied with the attachment tie (g) set to its maximum setting (figure 3). When raising furniture fitted with castors these should be removed and the castor adaptors (figure 2) positioned in their place. These can be screwed in place if necessary. The leg (f) or castor adaptor should be placed in the top cavity and the top adjustment ring (e) rotated relative to the body (c) to shorten the attachment tie (g) and retain the leg or castor adaptor (figure 1). The attachment tie is released by pressing the release button (d) and rotating the adjustment ring (e) in the opposite direction relative to the body (c).

Two castors that are close together can be raised with the MFR1.

Two MFR1 raisers can be placed side by side to take two castor adaptors that are close together such as on a divan bed. If the bases cannot be positioned close enough then cut off the marked section from the base of one or both of the MFR1 units (figure 4).

Do not move raised furniture that is occupied by a patient.

Maximum user weight including furniture is 500 kg (78 stone). Accepts legs up to 70mm (2.75 inches) diameter or 50mm (2 inches) square or equivalent rectangular or any other section.

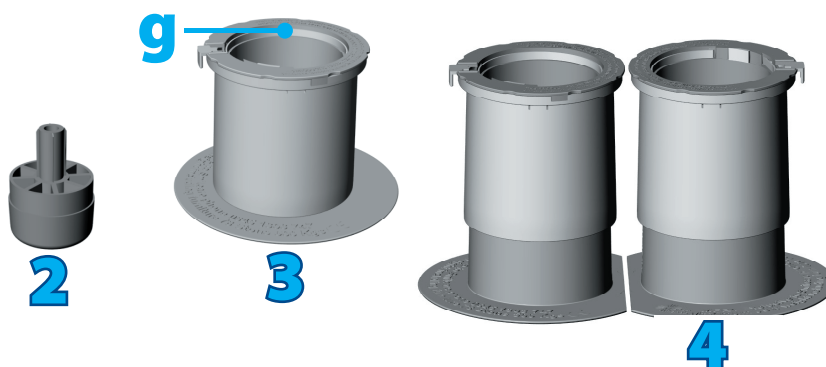
Smitcare Limited Unit 14 Camberley Business Centre
Bracebridge Camberley Surrey England GU15 3DP

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