Days 100 Series Lightweight Rollators Owner's Manual

ARTWORK
SIGN-OFF

Job No
71230

Is
3

Vendor
Wuxi Kangqiang Industrial
Equipment O Ltd

Date
12/09/19

PM
Beena

Studio Signature
AH

PM Signature

APPROVED

(please tick)

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GB Days 100 Series Lightweight Rollators
F Déambulateurs à roulettes légers Days série 100
DE Leicht-Rollatoren der Days 100 Serie
ES Andadores Days ligeros – Serie 100
I Deambulatori Leggeri Days Serie 100
NL Days 100 serie lichtgewicht rollators

| GB | F | DE | ES | I | NL |
|---|-----------------------------|-----------------------|---------------------------|----------------------------|----------------------|
| Model Numbers | Numéros des modèles: | Modellnummern: | Números de modelo: | Numeri dei modelli: | Modelnummers: |
| 102 / 09 135 6302 / 09 155 3023 / 09 155 3049 / 09 155 3056 / 09 155 3072 / 09 155 3082 | | | | | |
| 103 / 09 155 3098 / 09 155 3114 / 09 155 3122 / 09 155 3130 / 09 155 3106 / 09 155 3148 | | | | | |
| 105 / 09 135 6328 / 09 135 6336 / 09 154 6563 / 09 155 3163 / 09 155 3171 / 09 155 3155 | | | | | |
| 106 / 09 155 3189 / 09 155 3197 / 09 155 3205 / 09 155 3213 / 09 155 3221 / 09 135 6351 | | | | | |

| GB | F | DE | ES | I | NL |
|---------------------------|---|---------------------------|-----------------------------|----------------------------|-----------------------------|
| Available Colours: | Disponibles dans les coloris suivants: | Verfügbare Farben: | Colores disponibles: | Colori disponibili: | Beschikbare kleuren: |
| Ruby Red | Rouge rubis | Rubinrot | Rojo rubí | Rosso rubino | Robijnrood |
| Blue | Bleu | Blau | Azul | Blu | Blauw |
| Racing Green | Vert gazon | Racing-Grün | Verde inglés | Verde corsa | Donkergroen |
| Quartz | Quartz | Quartz | Cuarzo | Quarzo | Kwarts |
| Purple | Violet | Violett | Púrpura | Viola | Paars |
| Russet Orange | Rouge grenadier | Braun-Orange | Naranja rojizo | Arancio ruggine | Donkeroranje |

Days 100 Series Lightweight Rollators

Introduction

The Days Lightweight Rollator is a durable walker manufactured from painted aluminium tubing with four solid tyre wheels. It incorporates a rest seat, storage bag and locking brakes. This walker also has height adjustable handles and can be folded for storage or transportation.

Before using your new Days Lightweight Rollator, it is essential that you read and fully familiarise yourself with the instructions for use.

Specifications

| | 102 X-Small | 103 Small | 105 Medium | 106 Large |
|--------------------|---|--------------|---------------|--------------|
| Overall Height | 755-875mm | 790-910mm | 825-945mm | 875-995mm |
| Overall Width | 620mm | 605mm | 620mm | 620mm |
| Overall Depth | 640mm | 655mm | 655mm | 670mm |
| Seat Height | 505mm | 540mm | 545mm | 630mm |
| Seat Width | 350mm | 350mm | 350mm | 350mm |
| Seat Depth | 320mm | 320mm | 320mm | 320mm |
| Width between Arms | 460mm | 460mm | 460mm | 460mm |
| Backrest Height | 750mm | 800mm | 800mm | 870mm |
| Diameter of Wheels | 150mm | 200mm | 200mm | 200mm |
| Weight | 6kg | 7kg | 7.2kg | 7.3kg |
| Max. User Weight |  | | | |

Assembly Instructions

- Remove all packaging and examine all parts for signs of shipping damage. **Do not use Rollator if it is damaged or any parts are missing.**
- Open the Rollator frame fully. Push the folding hinge until straight to secure the Rollator in the correct position. Make sure all wheels make contact with the floor and the Rollator is stable. **(See Fig. 1)**
- Insert a handle with the pre-assembled locking loop brake into each handle tube. **Ensure that the brake cable runs down the outside of the frame. (See Fig 2)**
- Depress the pin clips on the backrest and slide it into the tubes at the front of the Rollator. You should hear a "click" as the pin clips lock into the securing holes. Ensure both sides of the backrest are in place with the pin clips protruding through the holes. **(See Fig 3)**
- Remove the protective plastic covering from the tyres.

Setting the Handle Height

- The handles have five settings at 25mm intervals.
- To determine the correct height of the handle **(See Fig 4)**;
 - Stand upright behind the Rollator with your shoulders relaxed and arms slightly bent at 15°.
 - Set the handle height to be level with your wrists
- To adjust each handle to the desired height, line up the holes appropriately. Slide the bolts through the holes from the inside of the frame and screw the tightening knobs until secure.

- Ensure that the bolt heads are countersunk within the holes, the tightening knobs are fully tightened and the handles are secure and stable.

When using the Rollator be aware that stability is dependent on all four wheels being in contact with a flat surface. Care should be taken when negotiating uneven ground, inclines and kerbs for example, as stability will inevitably be reduced.

Hand Brake Operation (See Fig 5)

The Rollator is fitted with locking brakes for added security. Before use test the brake by rotating the rear wheels and applying the brakes.

Note: Always engage and disengage brakes on both sides

- To apply the brakes, squeeze both brake handles.
- To lock the brakes, push both brake handles down until the click securely into the locked position.
- To release the brakes simply pull up on the brake handles until they click.

Adjusting the Brake Pressure (See Fig. 6)

If the brakes are too hard to apply, the pressure can be adjusted by reducing the tension in the spring above the brake shoe.

- The pressure needed to brake can be adjusted by turning the knurled knob whilst holding the hexagonal nut below it.
- Both the knob and nut are located above the brake shoe.
- Turning the knob anti-clockwise increases the pressure needed to brake by moving the brake shoe nearer to the wheel and compressing the spring.
- Turning the knob clockwise decreases the pressure needed to brake.

CAUTION

Brakes that are adjusted to give light pressure will be less effective at preventing rear wheel movement under load.

Using the Rest Seat (See Fig. 7)

- Before using the walker as a rest seat both brakes must be applied and the frame pushed forward to check that the brakes are fully engaged. The brakes only lock the back wheels so the frame will not be completely immobile.
- When seated place both feet firmly on the ground and do not lean backwards or sideways.
- Do not use the seat without the safety backrest
- Only use the seat on level surfaces
- Do not exceed the maximum user weight

Folding for Storage or Transportation (See Fig. 8)

The rollator is easily folded by lifting the seat and pulling the black strap attached to the hinge and the upper crossbar.

Pushing down on the handles unfolds the Rollator.

Safety Precautions

- Ensure the Rollator is fully open, the seat is down and the locking hinge is straight to secure the Rollator in the correct position
- Ensure both handles are at the same height and that the tightening knobs are fully tightened. Test the Rollator for stability before applying full weight to the handles.
- Test the brake mechanism before use. When the wheels are locked, the brake pad should prevent the wheels from turning. If the wheels turn while the brakes are locked, refer to **Adjusting the Brake Pressure** section. When the brakes are released, the wheels should turn freely.
- The Rollator is a walking aid only and should not be used as a transportation device. Do not allow anyone to sit on the Rollator while it is being pushed
- Do not leave the Rollator unattended without locking the brakes
- Do not use the Rollator as a seat in any type of motor vehicle
- Always put the brakes in the lock position before sitting on the Rollator
- Do not stop, turn or reverse on a ramp or incline
- Do not use the seat when the Rollator is on an incline or uneven ground
- Do not make sharp turns

Care and Maintenance

1. The Rollator should be inspected before each use to ensure it is free from damage and excessive wear.

Check that:

- The brakes are operating effectively
- The hand grips are not loose or damaged
- The brake cables are not damaged in any way
- The wheels are securely in place and are not damaged or excessively worn

- The frame is not cracked or dented
- The frame is secure, stable and all nuts and bolts are tight

If you do find wear or damage to your Rollator, DO NOT USE.

2. Check that all wheels and folding parts move freely and that there is no build-up of debris around the axle
3. Keep the Rollator clean by wiping with a damp cloth on a regular basis. Do not use abrasive cleaning materials
4. If required, only use genuine parts supplied by Performance Health International Ltd or an approved stockist

Warranty

Performance Health International Ltd warrants that the Rollator is free from defects in material and workmanship. The warranty shall remain in effect for **3 years** from the date of original consumer purchase.

The warranty does not extend to parts that are exposed to natural wear and tear that will require periodic replacement such as brakes, wheels, handgrips and bags.

This warranty does not cover the following:

Replacement parts supplied by anyone other than Performance Health International Ltd or the stockist;

Any failure of the unit during the warranty period if the failure is not caused by defect in material or workmanship or if the failure is caused by unreasonable use;

Performance Health International Ltd is not liable for incidental or consequential injury or damage to property.

Should it be necessary to return the rollator, it is essential that it is adequately packed to protect it in transit, preferably in its original packaging, otherwise Performance Health International Ltd cannot accept any responsibility for transit damage to the unit.

This does not affect your statutory rights.

Please retain these instructions for future reference.

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Days® is a registered trademark of Performance Health International Ltd.
Performance Health Supply, Inc.