2013-08-26 74274M

Etac Sverige AB
Box 203, 334 24 Anderstorp, Sweden
Tel 0371-58 73 00
Fax 0371-58 73 90
info@etac.se www.etac.se

Etac AB (export)
Box 203, 334 24 Anderstorp, Sweden
Tel 46 371-58 73 30
Fax 46 371-58 73 90
info@etac.se www.etac.com

Etac AS
Pb 249, 1501 Moss, Norge
Tel 815 69 469
Fax 69 27 09 11
hovedkontor.norge@etac.com www.etac.no

Etac A/S
Egeskovvej 12 8700 Horsens
Tel 79 68 58 33
Fax 75 68 58 40
info@etac.dk www.etac.dk

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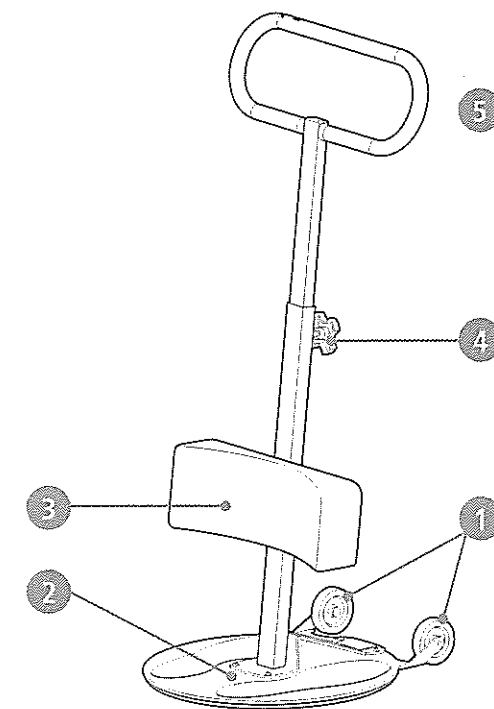
EN Reconditioning instruction

Turner is suitable for re-use. Perform the the cleaning and maintenance steps above. The turner must be transferred complete to the new user, including manual and other accompanying documents

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EN Maintenance

- Wheels / Wheel axles:** Check that the tyres, bearings and wheel axles are not worn, and that the wheels turn easily.
- Base plate:** Check that the base plate has no damage and that the anti-slip surfaces are not worn. Make sure the support bar is firmly anchored in the base plate. If the base plate is hard to rotate, wipe the dirt and grease between the sliding surfaces and lubricate again with Etac lubricant.
- Leg Support:** Check that the height adjustment works and is tightened. Make sure the pad does not have any damage.
- Height controls:** Check that the height controls work and are tightened.
- Handle:** Make sure the handle does not have any damage.
 - Keep the wheels and the space between the rotating plates free of dust, hair and suchlike. If required, wipe off any grease between the sliding surfaces and lubricate with silicone grease.
 - Check that the screws and controls are tightened properly. Perform inspection at least every 12 months.

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EN Safety

- Turner must not be used to transport people or be tipped while someone is standing on the rotating plate.

Guarantee:

5 year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com.
The expected life, during normal usage, is approximately 10 years. If your chair needs reconditioning or repair, use only original parts from Etac or components of similar quality.

Etac will not be held responsible for damage or injury caused by use of non-original parts.

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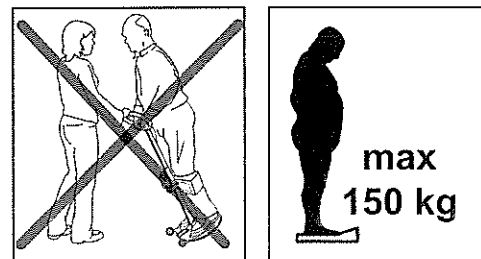
- EN** Turner is a sit-to-stand and transfer aid that makes it easier for a carer to help a user to stand and transfer him/her to another sitting position. Used by users who can grip and are able to pull up their own body weight, but still require support/help to stand up and be moved.

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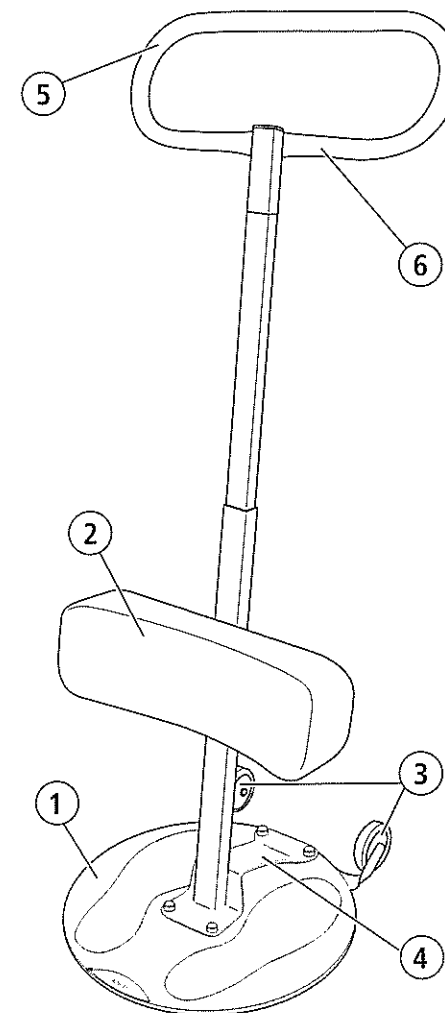
- EN** 1. Rotating plate
2. Leg support
3. Transport wheels
4. Brake-pedal
5. Handle
6. User handle

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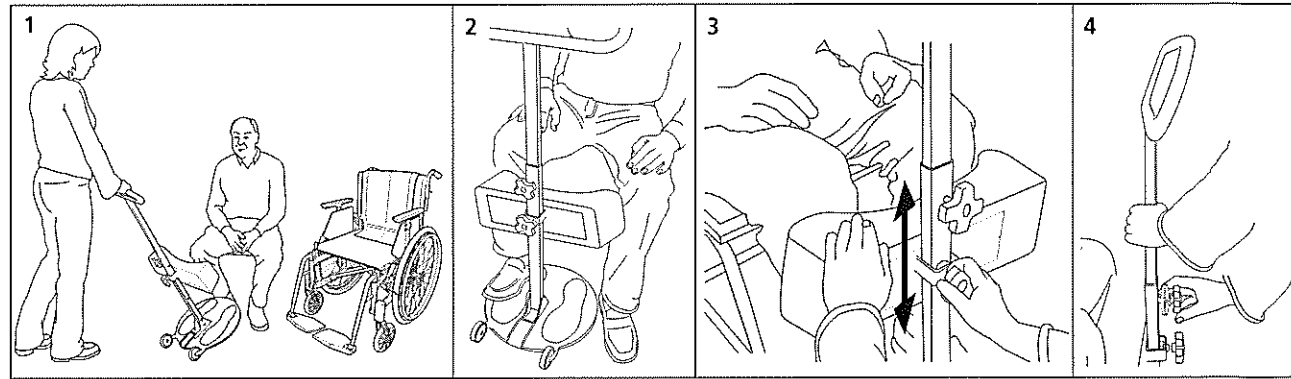
- EN** 1. Begin by positioning the new seat (chair, wheelchair, toilet or bed) in line with the existing seat.
2. Slide Turner in under the user's feet.
3. Adjust the height of the lower leg support so that its upper edge lines up just under the knee of the user.
4. Adjust the height of the handle to a position that is comfortable for both user and carer.
5. Grip the handle with both hands and place one foot on the pedal.
6. The user grips the handle, if possible with both hands.
7. Encourage the user to stand up while, at the same time, counterbalancing the user's weight using a foot and hands (use your bodyweight by leaning backwards).
8. Twist Turner so that the user is transferred to the front of the new seat.
9. Step on the pedal, brace your weight and ask the user to sit down. Maintain eye contact with the user during the entire transfer process.

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